

# HEMOSTATIC TOURNIQUET «Dnipro»

TY 32.5-34228792-001:2022

## DNIPRO



**Name of the medical product:**

Hemostatic tourniquet «Dnipro»  
\*hereinafter tourniquet

**Composition and description of the product:**

Hemostatic tourniquet of 94 cm length. Velcro fastener. A metal stop-lock (**turncoat**). Platform with clip. Metal buckle. **Top Velcro band**.

**Product is used for:**

Stopping critical bleeding on the upper and lower extremities. For single use.

**Warning:**

Use as intended.

**Application:**


For the provision of first aid at the medical institutions, ambulances, during hostilities and other emergencies.

**Storage conditions:**

Store in a clean, dry room with a relative humidity of no more than 80% at room, room temperature from 5°C to 25°C. Avoid direct sunlight.

**Expiration date:**

3 years from the date of manufacture. After unpacking, store in a closed first-aid kit or bag.

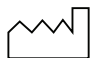 05/12/2022

TOURNIQUET «Dnipro»	
UA Patent №151561 U.S. Patent: Patent Pending EU Patent: Patent Pending	Manufactured by MC Prostrir LLC 49019, Ukraine, Dnipro, St. Akademika Belelyubskoho, 68



website: [www.tqdnipro.com](http://www.tqdnipro.com)  
email address: [priem@prostrir-e.com](mailto:priem@prostrir-e.com)




# INSTRUCTIONS FOR USING THE MEDICAL PRODUCT

1



Unfold the folded harness. Place the resulting loop on the Wounded Limb. Place the tourniquet as high as possible above the wound and perpendicular to the limb.

2

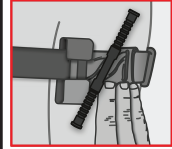

Tighten the tourniquet as tightly as possible around the limb and fix the Velcro

by pressing the upper one to the lower one.\***If the first turn of the harness is loosely applied and the velcro of the Fastener is weakly pressed, the correct operation of the product is not guaranteed.**

3

Turn the stopper several times until the bleeding stops and a pulse is felt. The direction of rotation of the stop-locker (turn) does not matter.



**\*According to the results of practical research, with perpendicular placement of the tourniquet in relation to the limb and a tight first turn to cover critical bleeding, it is recommended: on the**

**upper limbs - up to 3 half-turns, on the lower - up to 5 half-turns.**

4



Fasten the stop lock in the locking clip. Check for bleeding and pulse. If the bleeding has not stopped or a pulse is felt, make another half tourniquet or apply a second tourniquet above


and next to the first tourniquet. Check bleeding and pulse again.

5



Fasten the free end of the harness over the stop - fixator (turn) between the clips and attach the upper Velcro to the clip on top of the harness.

6

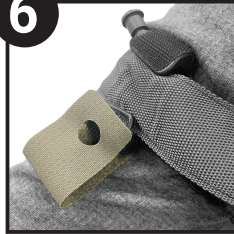

After fixing the upper velcro band on the clips, write on it the time of applying the harness. Prepare the victim for transport.

**USE OF ANY TOURNIQUET FOR MORE THAN 2 HOURS MAY RESULT IN IRREVERSIBLE NEUROLOGICAL OR MUSCULAR DAMAGE.**